



# GivMohr® SLING

Flaccid Upper Extremity Positioning Device  
Designed & Developed by Victoria Givler OTR and Paul Mohr PT

## Common Modifications to the GivMohr® Sling

- [Adding wrist splint for proper wrist position](#)
- [Adjusting handle for arthritic thumbs](#)
- [Decreasing handle volume to accommodate distal motor return](#)
- [Mix sling sizes to accommodate different body types and arm lengths \(Bilateral\)](#)

### Adding wrist splint for proper wrist position



### Adjusting handle for arthritic thumbs

#### Step #1

Place sling in warm oven (lowest setting) until handle is soft (5-10 minutes)

#### Step #2

Pinch handle and cool in COLD water





# GivMohr® SLING

Flaccid Upper Extremity Positioning Device  
Designed & Developed by Victoria Givler OTR and Paul Mohr PT


## Decreasing handle volume to accommodate distal motor return

### Step #1

Cut plastic handle along webbing edge

### Step #2

Wrap handle with mole skin to reduce handle size and pad rough edges.

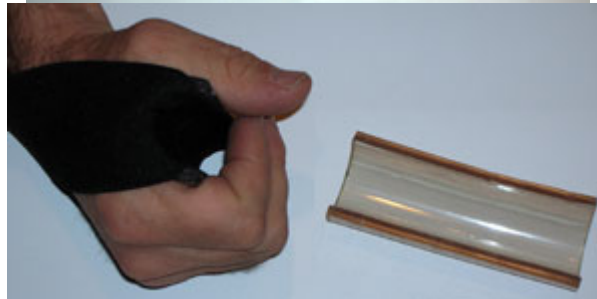

## Mix sling sizes to accommodate different body types

"Mix and Match" sizes of shoulder harness and elastic arm piece to accommodate unique sizes and/or different arm lengths (bilateral) of individuals

